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English

Product group:
Manual wheelchairs

Product:
Standard-Lightweight wheelchair

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The illustrations in these instructions may differ from the product supplied. Illustrations are only intended to aid understanding.

Thank you for choosing the move-it ONE standard lightweight wheelchair.

Please read these operating instructions carefully before using your new wheelchair. You will find important information that can help you when using the wheelchair.

In addition, you will receive information to ensure operational and road safety as well as the best possible maintenance of your standard lightweight wheelchair move-it ONE.

If you would like further information about your care, please contact your rehab dealer.

This user manual contains the necessary instructions for fitting and operating the move-it ONE wheelchair. The information cannot be transferred to other wheelchairs.

Introduction

Standard-Lightweight wheelchair

These instructions apply to the wheelchair with the article number UN-2014-xx

Intended use:

The move-it ONE wheelchair is designed for indoor and outdoor use. It is intended exclusively for self-propelled and third-party passenger transport.

The move-it ONE is designed and tested for a maximum load of 120 kg.

Description:

The move-it ONE is fully assembled at the factory and packed in a cardboard box. Please keep the packaging if possible, as it will be used to store the wheelchair at a later date.


Scope of delivery:

Please check whether you have received the complete wheelchair. The scope of delivery consists of

- 1 Outer packaging
- 1 Wheelchair pre-assembled
- 1 Pair of leg rests
- 1 Calf strap
- Operating instructions


General view

Front view: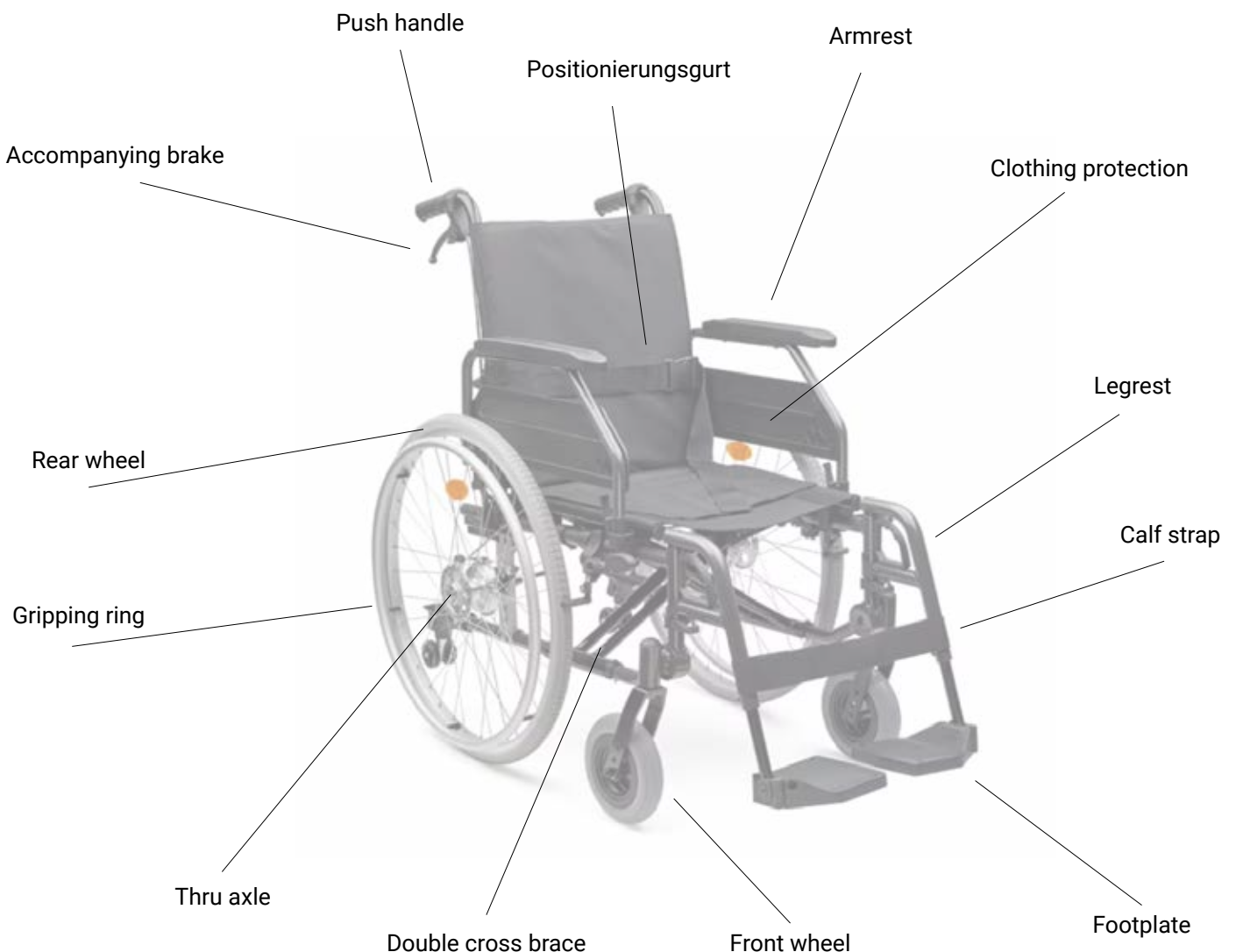

General view

Back view: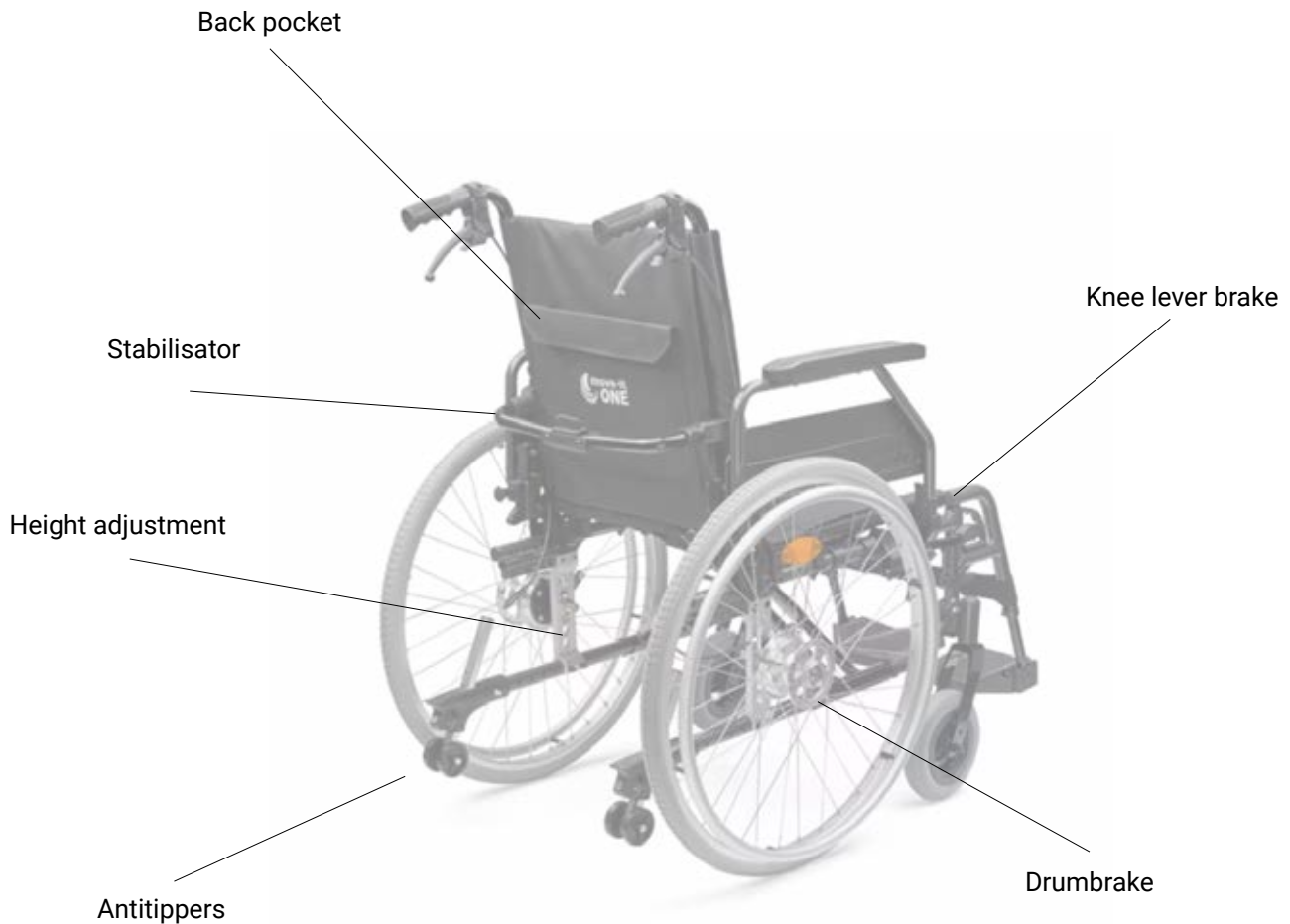

Structure


The wheelchair is completely pre-assembled ex works. All you have to do is take it out of the packaging and unfold it. During initial assembly, check all mounting screws, the correct fit of the attachments, and the brake settings.

Stand in front of the wheelchair and carefully pull the wheelchair apart a little. Then carefully press down on both sides of the seat until the wheelchair is completely unfolded.

Please note that the cross strut engages when sufficient pressure is applied. This is the sign that the wheelchair can now be used safely.

After unfolding the wheelchair, please secure the stabiliser on the back of the backrest.

Now align the calf strap and the foot plates with the patient for a comfortable fit.

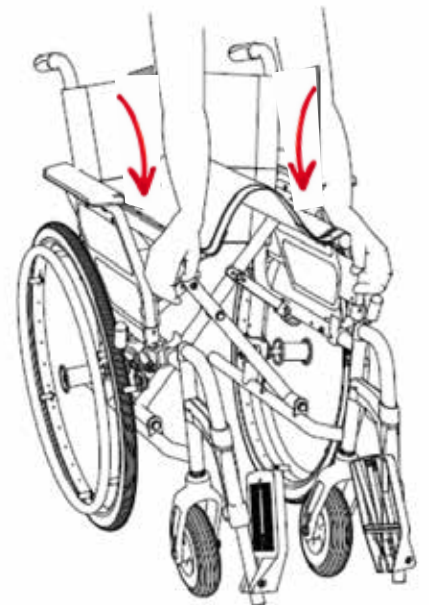Adjustment options

Swivel up the side panels

To swivel the side sections upwards and backwards, press the lock and pull the armrest at the front upwards.

Adjusting the footrests


The foot plates can be folded upwards to ensure access to the wheelchair.

The length of the footrests can also be adjusted to adapt the wheelchair to the user's height or leg length. To do this, loosen the locking screw and adjust the footrest to the desired length. Then re-tighten the locking screw securely.

Swivelling the footrest

To swivel the entire footrest sideways, pull the locking mechanism upwards and swivel the entire footrest outwards.

If you want to remove the complete footrest, pull the locking mechanism as described above and lift the complete footrest upwards.

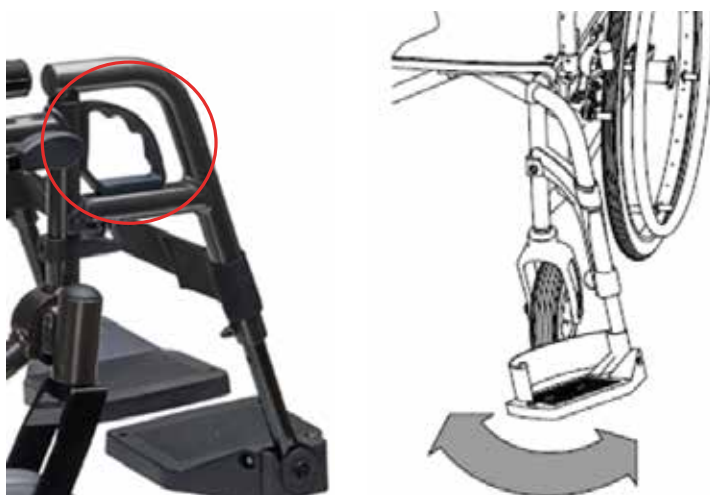Velcro Back covering

The back tension can be tightened and loosened using the Velcro back fitted as standard. This allows the backrest to be easily and comfortably adjusted to the user and an optimum sitting position to be achieved.

Adjustment of the seat depth, seat height and backrest angle

This wheelchair can be ideally adapted to a patient using various adjustment options. However, the weight distribution and handling of the wheelchair can change fundamentally as a result of the adjustment. We therefore ask you to visit your specialist rehabilitation dealer for these adjustments.

Adjusting the backrest tilt

Adjusting the seat depth

Adjusting the front and rear seat height

Using a wheelchair

Sit down and stand up

If you are using the wheelchair without an attendant, place it backwards against a wall to sit down. This ensures that the wheelchair does not slide away on a slippery surface.

To sit down without an accompanying person, we recommend the following procedure:

Preparation:

Remove the calf strap, loosen the leg supports and swivel them outwards to allow access to the wheelchair.

Sit down:

If possible, position the wheelchair with the rear wheels against a stable support and/or lock the wheelchair with the knee lever brakes.

Stand as close as possible to the wheelchair, turn backwards towards the wheelchair and carefully move back until you feel the seat in your knees.


Now reach backwards with both hands and support yourself on the armrests.

Now sit down slowly and carefully.

As soon as you have adopted a comfortable sitting position, swivel the footrests inwards and secure them (the footrest engages with an audible click). Then fasten the calf strap so that you are comfortably positioned.

Attach the safety belt to stabilise the person sitting in the wheelchair. This prevents the wheelchair from tipping forwards.

Get up: Reverse the procedure to get up from the wheelchair.

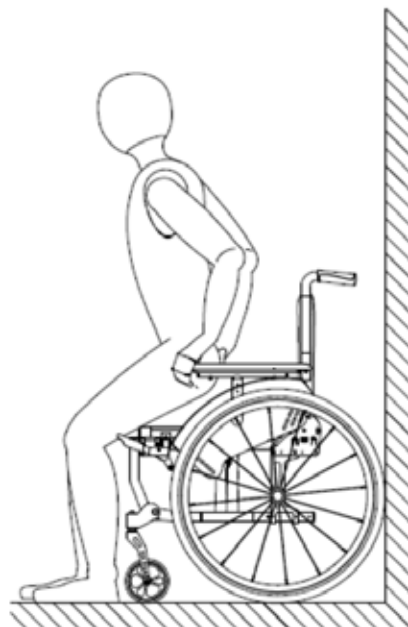Note:

Whenever possible, please have an accompanying person help you to sit down and stand up.

Wheelchair driving and braking

Specially manufactured push rims are fitted to the wheels. Now grip the push rims with your hand and use them to turn the wheel forwards or backwards.

Note:

When reversing, always check for any obstacles that may be behind you.

The thumb is at the top of the push rim, the other fingers are formed into a fist and thus grip the entire push rim.

To slow down, let the handrim slide through your hand in a controlled manner by applying pressure with your fist.

Note:

During prolonged braking manoeuvres, the push rims can heat up due to friction with the skin.

Note:

When moving the push rims forwards or backwards, take care not to touch the push rims with your hand. There is a risk of trapping between the tyre and the side section!

To steer the wheelchair in a controlled manner, repeat the same manoeuvre as for braking, but only on one side of the wheelchair (with one hand only). Slightly brake the wheel on the desired side to turn in this direction.

To turn in place, hold the push rim firmly on one side and move the wheel forwards or backwards on the other side.

Wheelchair parking with parking brake

Knee lever brake

Push the knee lever brake forwards to press the brake rod onto the tyre. Please note that prolonged pressure on the tyres can leave dents.

To release, push the brake lever upwards again.

Accompanying brake

Pull the escort brake upwards and lock it with the locking lever. To release, apply the brake and release it carefully.

Tipping aid with integrated anti-tipping wheels

The anti-tip wheels prevent the wheelchair from tipping backwards. They also make it easier for an attendant to tip the wheelchair due to the ribbing at the top.

The height of the anti-tip wheels can be adjusted. To do this, pull out the fixing pin, adjust the wheel to the desired height and refit the fixing pin.

The integrated tilting aid enables the attendant to tilt the wheelchair backwards with little effort and thus lift the front wheels over a threshold or similar.

Note:

Before tipping by an attendant, the anti-tip castors should be set to the highest position so as not to hinder the desired tipping.


Transport and folding

The wheelchair can be conveniently folded for transport in a car or for storage.

First remove the seat cushion (optional) and fold the foot plates upwards. Then take hold of the seat cover in the centre at the front and back and carefully lift it up. The wheelchair then folds up automatically.

Note:

Ask a specialist rehab dealer to show you how to use it - there is a risk of pinching your fingers.

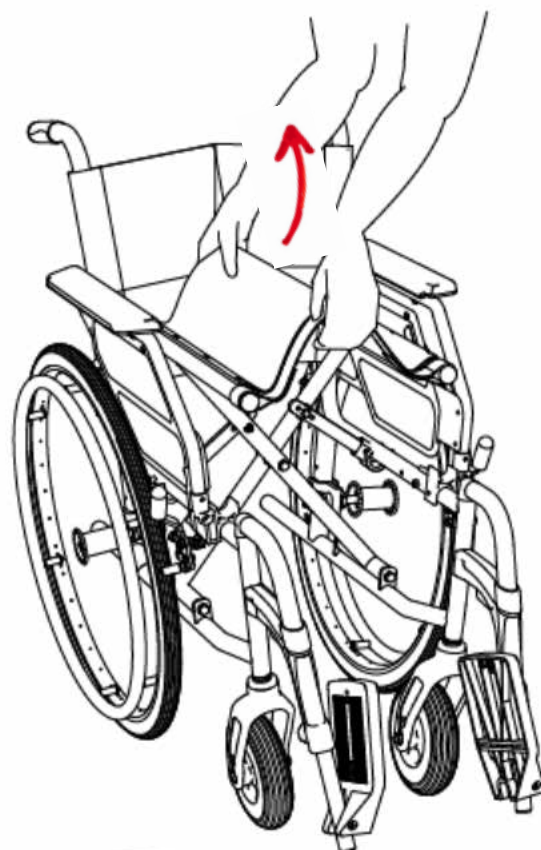Note:

Always open the locking lever of the stabilizer before folding.

Technical data and documentation

	37 cm	40 cm	43 cm	46 cm	49 cm	52 cm
	110 cm	110 cm	110 cm	110 cm	110 cm	110 cm
	59 cm	62 cm	65 cm	68 cm	71 cm	74 cm
	33 cm	33 cm	33 cm	33 cm	33 cm	33 cm
	95 cm	95 cm	95 cm	95 cm	95 cm	95 cm
	42cm 44.5cm 47cm	42cm 44.5cm 47cm	42cm 44.5cm 47cm	42cm 44.5cm 47cm	42cm 44.5cm 47cm	42cm 44.5cm 47cm
	45 cm	45 cm	45 cm	45 cm	45 cm	45 cm
	47cm 49cm 51cm	47cm 49cm 51cm	47cm 49cm 51cm	47cm 49cm 51cm	47cm 49cm 51cm	47cm 49cm 51cm

Materials

Frame with double folding cross made of aluminium
 Frame aluminium powder-coated
 Rear wheels 24", PU, puncture-proof
 Front wheels 7", PU, puncture-proof
 Accompanying brake with drum brake
 Knee lever brakes
 Tilt aid with integrated
 anti-tilt castors
 Swivel-away footrests
 Fold-up footrests
 Swivel-up and removable side parts
 Positioning belt
 Weight of wheelchair: 16.5-19kg
 Tested maximum weight: 120kg

Disposal instructions

Note:

If you have been loaned the wheelchair for use, follow the instructions of the organisation that provided you with the wheelchair.

Note:

Observe the applicable disposal guidelines in your municipality.

Dispose of the wheelchair via a waste disposal company or a public waste disposal centre in your region. The dealer who sold you the wheelchair can also dispose of it. Please note that disposal may be associated with costs.

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CE Declaration of Conformity
According to (EU) 2017/745 MDR

DECLARATION OF CONFORMITY
ACCORDING TO (EU) 2017/745 MEDICAL DEVICE REGULATION

EU Representative

Lotus NL B.V
Koningin Julianaplein 10,le
Verd,2595AA, The
Hague,NetherlandsAmsterdam,
Netherlands
SRN: NL-AR-000000121

Conformity Assessment

Conformity Assessment Procedure
Annex II+III of Regulation (EU) 2017/745

Applicable Standards
EN ISO 14971:2019
EN ISO 15223-1:2021
EN 1041:2008+A1:2013
ISO 10993-1:2018
EN ISO 10993-5:2009
EN ISO 11199-2:2021

Manufacturer

Name:
Zhenjiang U-Nurse Medical Equipment Co., Ltd.

Operation Address: Building 16,Central R&D
Zone,Science and Technology Industrial
Park,Development Zone,No99, Jing15th Road,New
District,Zhenjiang City,JIANGSU Province,China

Registered Address:Building 2,Central R&D
Zone,Science and Technology Industrial
Park,Development Zone,No99, Jing15th Road,New
District,Zhenjiang City,JIANGSU Province,China

Product Information

Name: ALUMINUM WHEELCHAIR
Model:UN2001,UN2002,UN2003,UN2004,
UN2005,UN2006,UN2007,UN2008,UN2009,
UN2010,UN2011,UN2012,UN2013,UN2014,
UN2015,UN2016,UN2017,UN2018,UN2019,UN2020,UN2030,
UN2031,UN2032,UN2033, UN2034, UN2035.

SRN: CN-MF-000013494

Basic UDI-DI: 697479546 wheelchair1006Q
Classification: Class I, According to Rule 1, Annex
VIII, Regulation (EU) 2017/745

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You will receive a purchase receipt from your specialist dealer. Please transfer the information received to this warranty certificate and keep it together with the operating instructions.

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