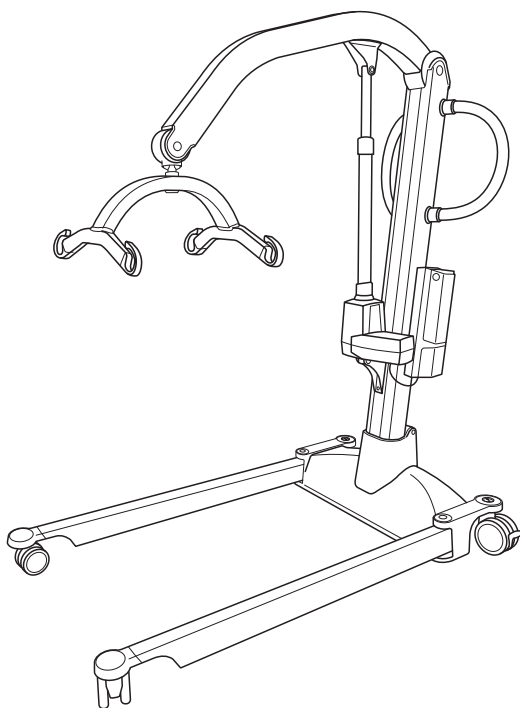
# User manual

Molift Mover 205

**molift**<sup>®</sup>  
by Etac

BM05299 - 2025-05-20 - Rev. 2.0

[www.etac.com](http://www.etac.com)

# CONTENT

## Symbols

3

## Device overview

9

## General

12

English

12

Svenska

21

Norsk

30

Dansk

39

Suomi

48

Deutsch

57

Nederlands

66

Français

75

Italiano

84

Español

93

Čeština

102

Slovenščini

111

Polski

120

## Handling

129

Assembly

130

Warnings

132

Information

134

Checklist

136

Emergency stop

138

Warnings

140

Information

142

Rising

144

Lowering

146

Gate training – rising

147

Gate training – lowering

149

Warnings

150

Information

155

## Technical data

157

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## General

Thank you for choosing an Etac device.

This manual must be read thoroughly before you start using the device as it contains important safety information and gives instructions on how to use the device correctly. Visit [www.etac.com](http://www.etac.com) for all relevant product documentation, for example the user manual, maintenance and recycling instructions. You can select your language via the “International” and “Local websites” link.

You can also consult the QR code on the device for further information and instructions.

### Copyright

This document is the property of Etac A/S. The content shall not be modified in whole or part without prior approval of Etac A/S.

### Disclaimer

Etac continuously makes improvements to our devices and information. We therefore reserve the right to modify our devices and instructions for use without prior notice. Consult [www.etac.com](http://www.etac.com) to find the latest version of the documentation.

### CE Declaration of Conformity

The device mentioned in this manual and the different models hereof comply with the Medical Device Regulation (EU) 2017/745.

The device is classified as a Class I medical device.

### Product standards

The device has been tested and complies with the requirements applicable for this type of device in the below mentioned standards:

- ISO 10535:2021 Assistive products – Hoists for the transfer of persons – Requirements and test methods.
- IEC 60601-1:2005 Medical electrical equipment – Part 1: General requirements for basic safety and essential performance.

- IEC 60601-1-2:2014 Medical electrical equipment – Part 1-2: General requirements for basic safety and essential performance – Colateral Standard: Electromagnetic disturbances – Requirements and tests.

The product has been tested and approved by a third party according to standards IEC 60601-1, IEC 60601-1-2 and EN/ISO 10535:2021.

### Warranty

Please refer to [www.etac.com/support](http://www.etac.com/support).

If the customer makes adjustments, repairs or uses combinations not predetermined by Etac, the Etac CE certification and Etac warranty will not be valid. If in doubt, please contact Etac.

### Notice

Any serious incident that has occurred in relation to the device should be reported to your distributor/representative and the national competent authority within your region in a timely manner. The distributor/representative will forward the information to the manufacturer.

### Pre-sales information

Please visit [www.etac.com](http://www.etac.com).

## Device description

Molift Mover 205 is a mobile hoist intended for hoisting and transferring a person from/to bed, floor, chair, wheelchair or toilet with a sling. The hoist is made of light-weight materials, has castors and electric actuator and is powered by a battery. It has electrically adjustable legs and a lifting capacity of 205 kg (452 lbs).

Molift Mover 205 should not run continuously for more than 2 minutes (with max load). After 2 minutes the hoist needs to rest for 18 minutes. Duty cycle 10%. (Intermittence according to standard ISO-EN 10535). The device has a maximum A-weighted sound power level: Lwa = 63.2 dB.

It is ideal for institutional care settings but can also be used in private homes and nursing homes. The hoist should only be used for transfers across short distances, such as between a chair and a bed or into bathroom/rest room, and is not a replacement for wheelchair or similar.

### Benefits

The clinical benefits of Molift Mover 205 are:

- Reduced load or strain on carer.
- Reduced risk of work-related injury.

### Materials

Aluminium, plastic, steel.

## Introduction

The person referred to as “the user” is the person who is sitting in the product. “The carer” is the person who manoeuvres the product.

### Intended purpose

The mobile hoist is an assistive device intended for alleviation of, or compensation for, a functional impairment due to an injury or disability. The device is designed for an individual lacking the ability to stand up and transfer themselves to/from a bed, the floor, a wheelchair, a chair, a toilet or similar due to reduced mobility or physical strength.

### Intended user

The device can be used for children, adults and elderly people with a need for support to be lifted and transferred over shorter distances or when gait training.

### Intended environment

For indoor use on horizontal surfaces in acute care, long term care and home care.

### Indications

For users who are in need of support to be lifted and transferred over a shorter distance or during gait training; e.g. people with limited ability to move and who require body support during transfers. Not an exhaustive list. The user group for the device is based on individual health and mobility function and not on a specific diagnosis or age.

### Contraindications

There are no known contraindications.

### Precautions

Make sure the sling fits the user and the sling bar.

### Training requirements

Only carers who have read and understood the user manual are allowed to use the device.

## Safety information

The safety of both user and carer must never be jeopardized – if you suspect that the device has a malfunction you must put it in quarantine marked “out of order” and call your local distributor/representative.

Safety signs, safety measures and warnings symbols are given in the context relevant for the actual handling and/or operating situation – it is mandatory the given instruction is followed to avoid dangerous/hazardous situations.

Signs, symbols and instructions placed on this device must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the device. Immediately replace or repair illegible or damaged signs, symbols, and instructions. Contact your distributor/representative for instructions.

### Patient transfer

Transfer of a user will always pose a certain risk. Therefore, a risk assessment must always be carried out before the transfer is performed.

Warnings:

- Never use the device for transportation of goods.
- Never use the device for self-transportation.
- Never leave a user unattended in the device.
- Never use the device in the shower.
- Always transfer the user in a calm and stable speed.
- Never transfer the user in a standing position.

### Risk assessment

Do a risk assessment of the device, the individual user and the transfer situation, and document your analysis. You as the carer are responsible for the safety of the user.

Plan the transfer in advance to ensure that it is performed as safe and smooth as possible.

Remember to work ergonomically according to local recommendations.

The carer has to be able to handle the weight of the user.

To avoid serious injury, the user shall avoid extended reaching or leaning movements.

## The device

Always check – before each use:

- that the device is suitable for the user and the transfer situation.
- that the device does not have visible damage.
- that the sling bar connection and all detachable parts are properly connected and secured.
- that the service LED (LED 4) is not on and the battery indicator has at least one green LED.
- that the sling strap does not have visible damage or frays.
- that the wheels are rolling freely and hoist is easy to maneuver.

Inspection to be performed monthly:

- test run hoists operation functions and make sure hoist does not make any abnormal sounds.
- test emergency stop button and emergency lowering.

If the inspection reveals any safety risks, the device must immediately be taken out of service.

## Materials

All used materials which are to be in direct contact with skin are biologically assessed and evaluated according to relevant standards. However, users/carers who are allergic to any of the materials are recommended not to use the device.

## Accessories

User manuals supplied with accessories must be read and understood before use. This includes any limitations in use and combinations.

Information about accessories can be found on [www.etac.com](http://www.etac.com).

## Spare parts

A list of spare parts is available at [www.etac.com](http://www.etac.com).

## Device combinations and sling compatibility

When combining a medical device with other devices or accessories, the combination must not change the by manufacturer defined intended purpose of the devices or modify the devices in such a way that compliance with applicable requirements, or safe use of the device are affected.

We recommend using Etac or Molift slings. The hoist can be combined with slings from other manufacturers, visit [www.etac.com](http://www.etac.com) for compatibility statements.

The person /company responsible for the combination of the devices must ensure that the applicable requirements are fulfilled.

Electric protection in combination with slings: Type BF Applied part.

If safe working load (SWL) differs between hoist, sling bar and sling, the lowest load should always be used.

## Safety measures

Molift Mover 205 is equipped with several safety devices, which are intended to prevent damage or injuries to personnel and equipment in the case of incorrect use.

The hoist has an overload sensor preventing the hoist to be operated if the load exceeds SWL.

The lifting arm is hinged to prevent the arm and sling bar from squeezing the user. The actuator will not produce any force when moving downwards.

The electronics are disabled if overheated. Wait until the hoist has cooled down before using it again.

## Electronics

### Battery

Molift Mover 205 has a 25.2 V lithium-ion NMC 2.85 Ah battery. The battery has a life expectancy of approx. 1700 charging cycles. Number of lifts with fully charged battery: 130 lifts (75 kg/165 lbs, 50 cm/20 inches).

Recharge battery upon receipt, and then every twelve months, to avoid battery failure.

Any damaged control box, battery or electronics on the device must be replaced.

Used batteries must be disposed of as special waste according to local rules and regulations. Do not dispose in general waste.

### Speed boost

The hoist comes with a speed boost feature that enables increased speed, up to 30% faster, when the actuator is unloaded.

### Motor/actuator

Lifting motor: 24 V DC.

Leg adjustment motor: 12 V DC.

Lifting speed: 40 mm/second (1.57 inches/second) with 75 kg (165 lbs) load.

### Battery and service indicator (LED)

The control box has five light indicators.

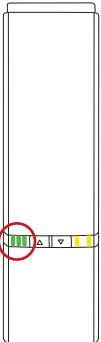
The electrical system has a power save function which will turn off the electrical system after two minutes without activity. All lights will turn off.

The system is activated when pushing one of the operating buttons.

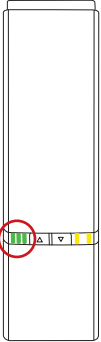
### Battery indicator (LED)

The battery indicator (LED) will illuminate and make a sound when battery level is low and the hoist requires charging. When this occurs the hoist will have sufficient power available for at least one full lifting cycle with max load.

## States in normal use

| LED indicator                                                                    | LED1 - LED2 - LED3                                                                | LED state<br>(Not listed = off)                                   | States                                                     |
|----------------------------------------------------------------------------------|-----------------------------------------------------------------------------------|-------------------------------------------------------------------|------------------------------------------------------------|
|  |  | LED 1-3 constantly on                                             | 75-100 % State of charge                                   |
|                                                                                  |  | LED 1+2 constantly on                                             | 50-75 % State of charge                                    |
|                                                                                  |  | LED 1 constantly on                                               | <50 % State of charge                                      |
|                                                                                  |  | LED 1 left side, switches from green to yellow and flashes slowly | Critically low battery (sound occurs when hoist is in use) |

## States while charging

| LED indicator                                                                     | LED1 - LED2 - LED3                                                                  | LED state<br>(Not listed = off)               | States                                                                                              |
|-----------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-----------------------------------------------|-----------------------------------------------------------------------------------------------------|
|  |    | LED 1-3 constantly on                         | 90-100 %                                                                                            |
|                                                                                   |    | LED 1+2 constantly on<br>LED 3 flashes slowly | 65-90 %                                                                                             |
|                                                                                   |    | LED 1 constantly on<br>LED 2 flashes slowly   | 40-65 %                                                                                             |
|                                                                                   |   | LED 1 flashes slowly                          | 0-40 %                                                                                              |
|                                                                                   |  | LED 1+2+3 flash slowly                        | Charging stopped due to low battery temperature, high battery temperature or other error conditions |
|                                                                                   |  | No light in LEDs                              | Charging stopped due to lost communication to battery                                               |

## Service indicator

| LED indicator                                                                    | Priority | LED4 + LED5                                                                       | LED state<br>(Not listed = off)                                               | States in normal use                         | Comments                                    | Reset                                                |
|----------------------------------------------------------------------------------|----------|-----------------------------------------------------------------------------------|-------------------------------------------------------------------------------|----------------------------------------------|---------------------------------------------|------------------------------------------------------|
|  | 0        |  | LED 4 flashing according to Bluetooth low energy pairing state                | Pairing Bluetooth low energy                 | Not ready to drive                          | Wait until ready                                     |
|                                                                                  | 1        |  | LED 4+5 constantly on                                                         | Emergency stop activated                     | Not ready to drive                          | Reactivate emergency stop                            |
|                                                                                  | 2        |  | LED 4+5 flashing fast (synchronous)                                           | FATAL ERROR<br>Cannot drive, has to be reset | No movement possible                        | Reset fatal error routine                            |
|                                                                                  | 3        |  | LED 4+5 flashing slowly (asynchronous toggling)                               | Not learned/configured correctly             | Not ready to drive                          | Contact local representative for service             |
|                                                                                  | 4        |  | LED 5 flashing slowly                                                         | OVERLOAD on motor/actuator                   | Momentary not ready to LIFT                 | Reduce load                                          |
|                                                                                  | 5        |  | LED 4 flashing slowly                                                         | Duty cycle guard                             | Momentary not ready to LIFT                 | Wait until ready                                     |
|                                                                                  | 6        |  | LED 4 slowly turns on and off                                                 | Periodic inspection needed                   | Drive is possible                           | Contact local representative for periodic inspection |
|                                                                                  | 7        |  | LED 4 constantly on                                                           | Service needed                               | Drive is possible                           | Contact service technician                           |
|                                                                                  | 8        |  | LED 4 constantly on (all remaining LED's flash while hoist is being operated) | Service overdue                              | Drive is possible (beeping while operating) | Contact service technician                           |

## Charging

The internal charger in the control box makes charging of the batteries efficient. Connect the power cable to the power socket located in the bottom corner of the control box.

The charging time depends on the battery capacity and condition. Typically, the charging time is approx. 3.5 hours from fully discharged state.

## Protection against electric shock (Insulation Class)

Internally powered during use/operation.

Class II during charging.

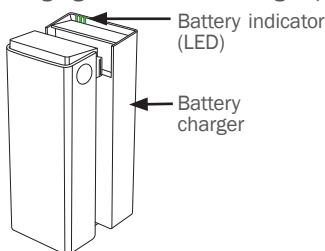
Hoist will not operate while charging.

## Hand control - Protection class: IPX6

| Overview                                                                          | Icons                                                                             | Description                                                                                                                                                                                         |
|-----------------------------------------------------------------------------------|-----------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  |  | LED state of battery, service and overload.                                                                                                                                                         |
|                                                                                   |  | Buttons for lifting and lowering of lifting arm.                                                                                                                                                    |
|                                                                                   |  | Buttons for leg adjustment mechanism in and out.<br>Fold-down feature: Press both buttons at the same time to lower the lifting arm and bring the legs together (feature activates after 1 second). |

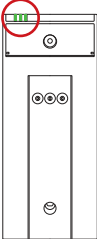
### Charging with external charger (Optional)

### Battery Charger

The battery charger can be used as a tabletop charger or mounted to a vertical wall surface close to a power outlet, minimum 120 cm above the floor. The charger has to be used outside the patient environment. The charger must be placed or installed in a way that makes it easy to disconnect mains cable plug. Take care not to damage the cable. The charger can be connected to the power outlet at all times.

Battery charger: 100-240 V AC, 50-60 Hz.  
0,9 A output.

| LED indicator                                                                     | LED1 - LED2 - LED3                                                                  | LED state (Not listed = off)                  | States while charging                                                                               |
|-----------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-----------------------------------------------|-----------------------------------------------------------------------------------------------------|
|  |    | LED 1-3 constantly on                         | With battery or if no battery mounted on charger: 90-100 %                                          |
|                                                                                   |    | LED 1+2 constantly on<br>LED 3 flashes slowly | 65-90 %                                                                                             |
|                                                                                   |  | LED 1 constantly on<br>LED 2 flashes slowly   | 40-65 %                                                                                             |
|                                                                                   |  | LED 1 flashes slowly                          | 0-40 %                                                                                              |
|                                                                                   |  | LED 1+2+3 flash slowly                        | Charging stopped due to low battery temperature, high battery temperature or other error conditions |
|                                                                                   |  | No light in LEDs                              | Charging stopped due to lost communication to battery                                               |

## Service, cleaning and maintenance

### Maintenance and service

Maintenance and inspections of the device is the responsibility of the owner of the device. Failure to maintain a device in accordance with the instructions may compromise the safety of the user and/or carer and may affect the expected lifetime of the device.

Inspection of the device must be carried out annually.

Check:

- that the device does not have visible damage.
- that labels are readable.
- that the leg adjustment and wheels move smoothly.
- that the brakes work.

### Service scope

Service must be carried out by authorized personnel.

Service is needed when service LED (LED 4) is on. The service LED (LED 4) turns on after a certain number of lifts or after 5 years, whichever comes first.

### Expected lifetime

The hoist has an expected lifetime of 30,000 cycles or 15 years, whichever comes first.

### Cleaning

Clean on a regular basis. Clean surfaces with a damp cloth using an appropriate pH-neutral detergent. Do not use solvents or strong liquids, this may damage surfaces on the hoist. For disinfection when needed; use isopropyl alcohol or similar. Avoid abrasive cleaning products. Remove hair and pile from the castors and verify that the castors rotate freely. Clean contact and hand control with isopropyl alcohol to remove grease and dirt. Check emergency stop and emergency lowering after cleaning. The hoist should not be exposed to running water.

Remove hair and filth from the wheels to ensure the wheels rotate freely and the brake functions properly.

Make sure not to damage or remove labels when cleaning.

### Reconditioning

Separate reconditioning document is available at [www.etac.com](http://www.etac.com).

### Periodic inspection

Periodic inspection shall be performed annually or more frequently if required by local requirements. See the periodic inspection label on the product and the periodic inspection document at [www.etac.com](http://www.etac.com).

## Transportation, storage and disposal

### Transportation and storage

When the device is not in use, it can be stored under the following conditions:

Keep packaging away from children.

Keep away from fire.

Medical electrical equipment requires special precautions regarding electromagnetic compatibility (EMC). Interference may occur in the vicinity of the hoists electronics.

For long time storage it is recommended that the emergency stop button is activated (pushed in). The hoist can be stored and transported under temperatures between -25 to 70 °C and relative humidity 15 - 93%.

Air pressure: 70 - 106 kPa.

### Disposal

The device contains no hazardous substances that require special disposal. Check if there are options available for recycling in an environmentally friendly way and follow the national provisions that apply. Dispose of the device according to the national provisions that apply.

### Recycling

See the recycling instructions on [www.etac.com](http://www.etac.com), and follow the national provisions that apply.

## Troubleshooting

| Problem                                                                                  | Action                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                             |
|------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| The lifting column is unstable.                                                          | The lifting column is not correctly positioned in the chassis or the locking handle is not fully locked. Assemble column and chassis properly.                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                     |
| The hoist moves unevenly on a plane surface.                                             | The castors move unevenly because of pile and dust.<br>Clean the castors. Remove hair and dirt.                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                    |
| The lifting arm or adjustable legs moves uncontrolled.                                   | Short circuit between hand control and hoist.<br>Clean hand control plug surfaced with alcohol to remove grease.<br>Defective hand control. Replace hand control.<br>The control electronics has failed. Contact local representative for service.                                                                                                                                                                                                                                                                                                                                                                                                                 |
| The hoist does not respond to hand control action/the lifting arm or legs does not move. | Emergency stop button is activated. To release emergency stop take off battery and replace again.<br>Battery is discharged. Change or charge battery.<br>The hoists electronics is overheated. Wait for it to cool down.<br>The hoist is overloaded. Make sure the load does not exceed SWL.<br>Hand control is not plugged in properly. Plug in hand control properly.<br>Hand control, plug or cord can be broken and should be replaced.<br>Contact between chassis and column has no connection. Column is not correctly mounted in chassis. Assemble column and chassis properly.<br>Defective control electronics. Contact local representative for service. |
| Battery does not charge.                                                                 | Faulty battery. Check indicators on charger when charging. Try another battery.<br>Faulty battery charger. Check indicators on charger when charging. See chapter "Charging". Try another charger.                                                                                                                                                                                                                                                                                                                                                                                                                                                                 |

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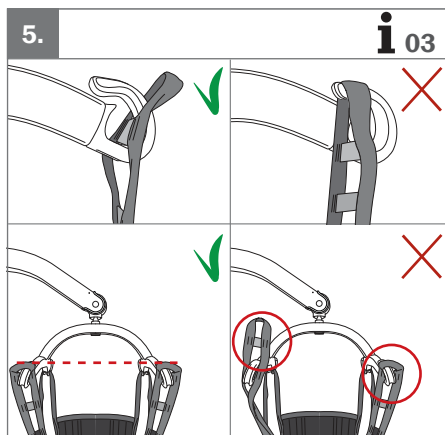
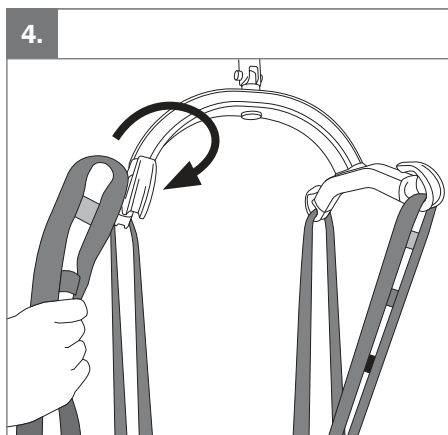
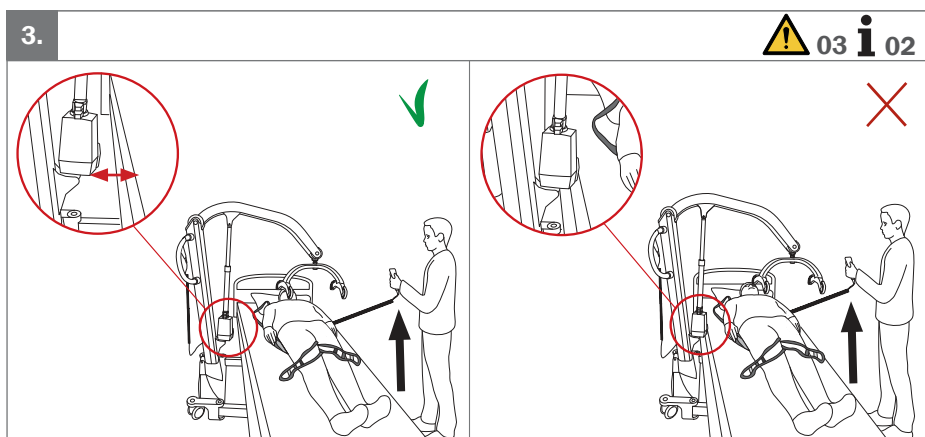
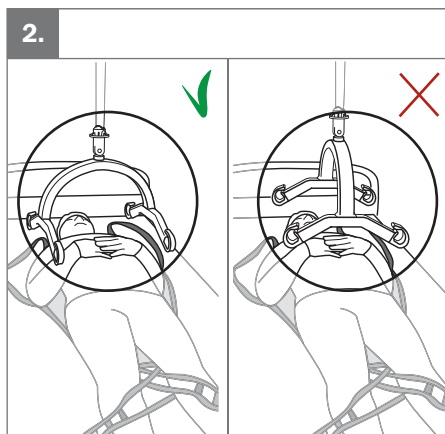
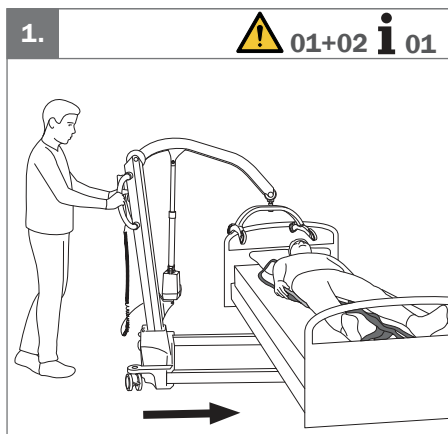
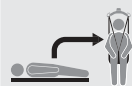
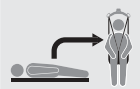
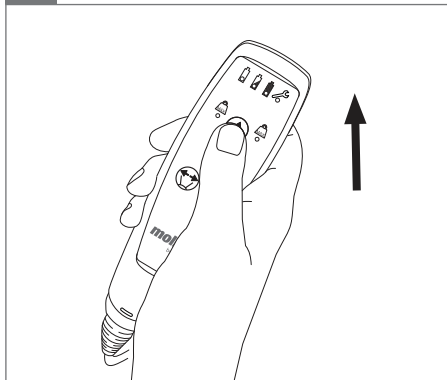
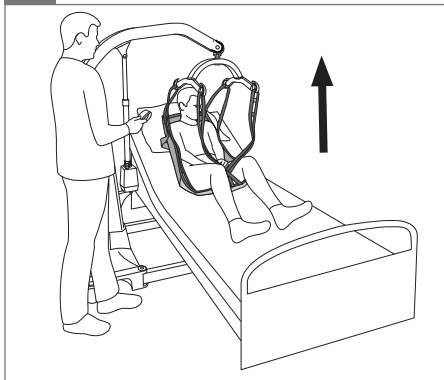
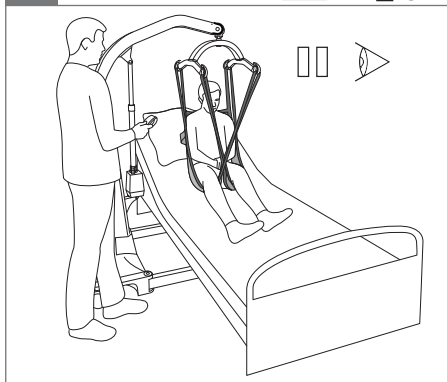
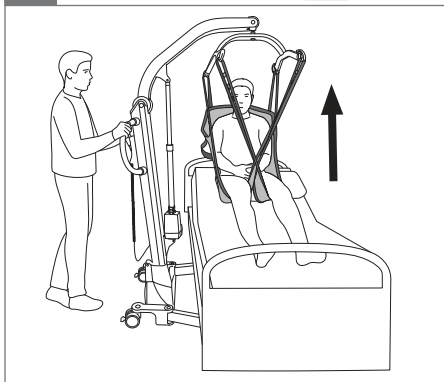
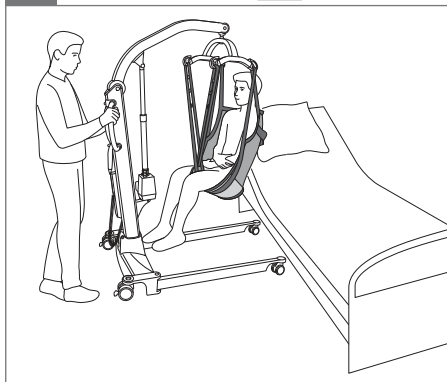
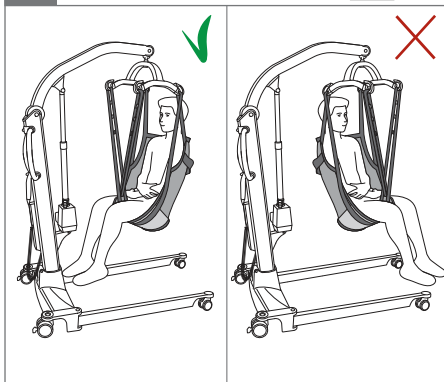
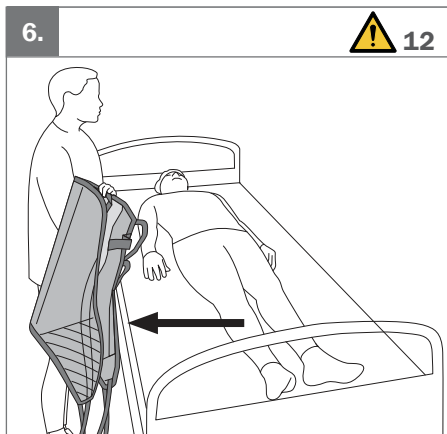
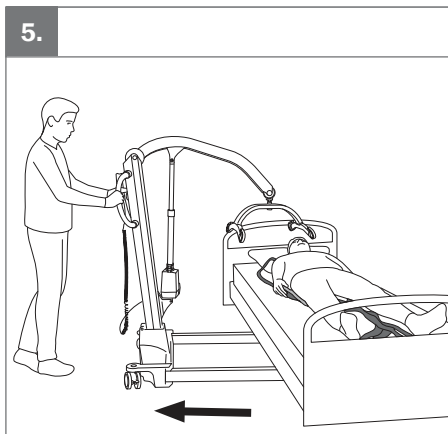
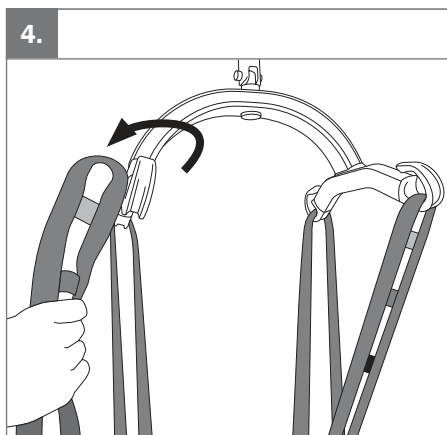
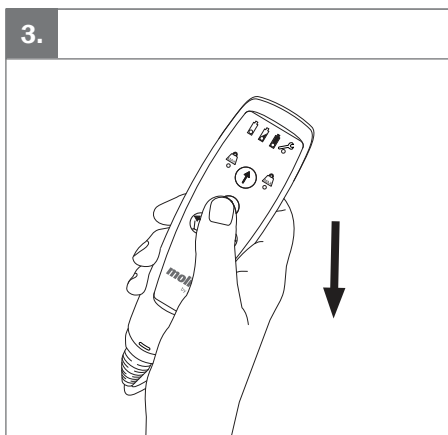
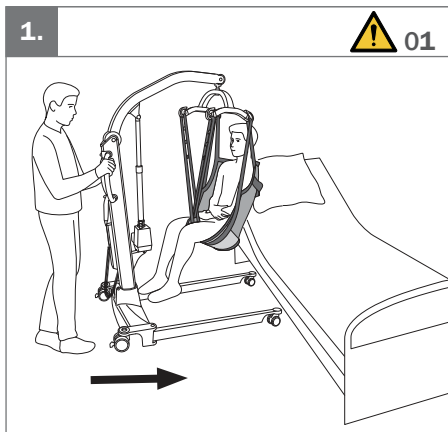
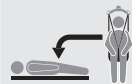
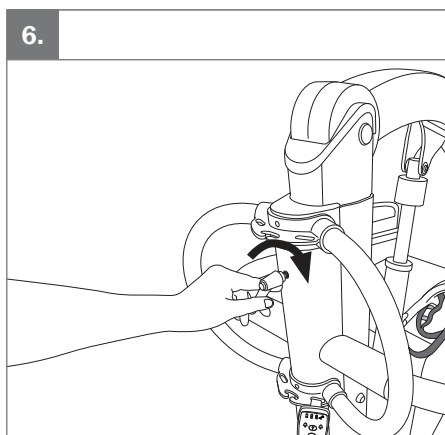
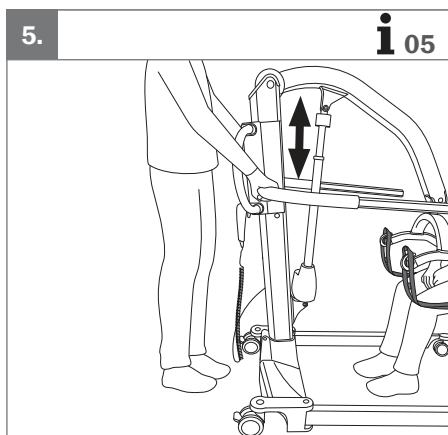
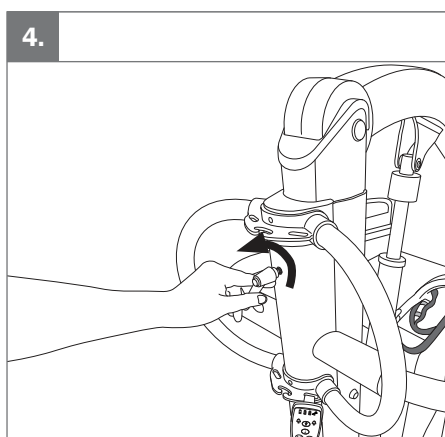
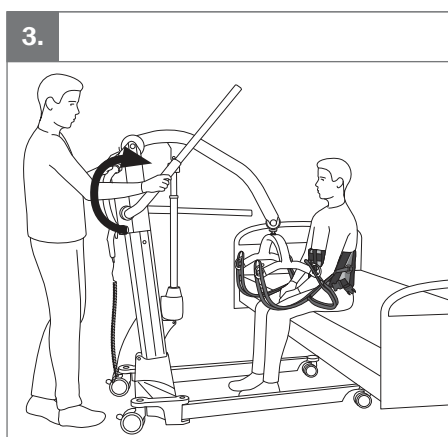
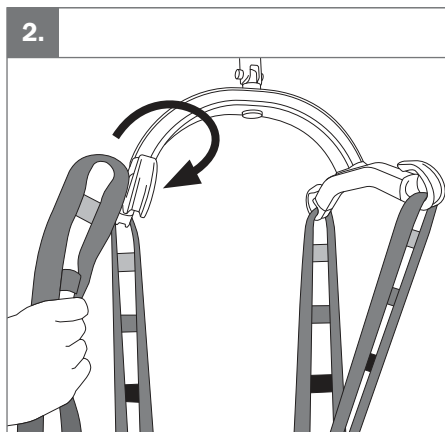
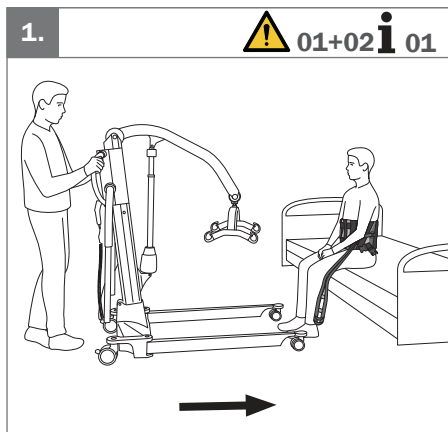
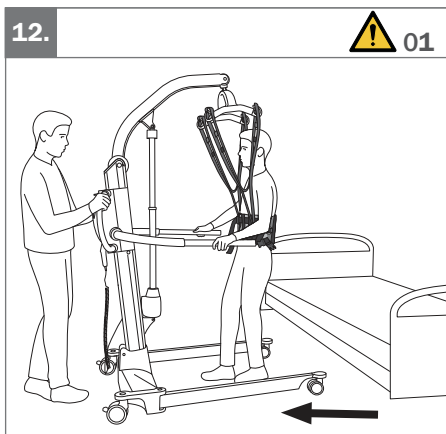
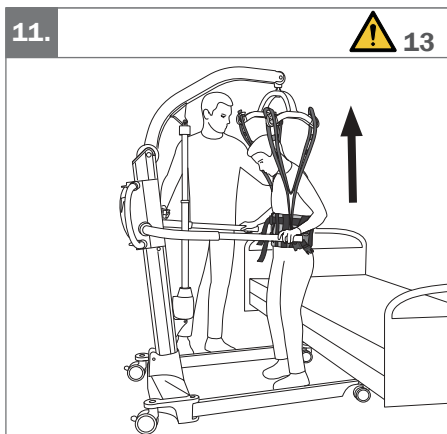
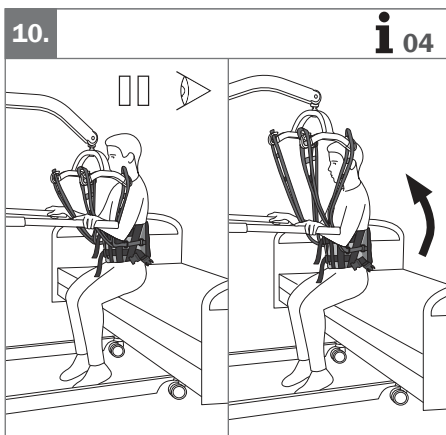
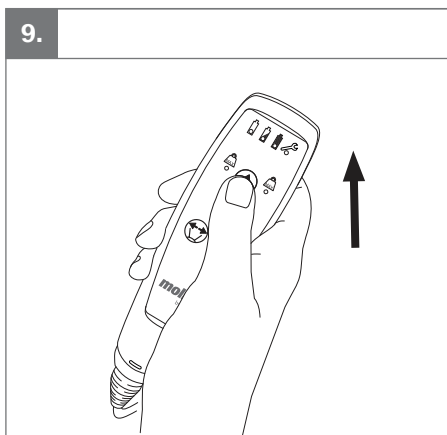
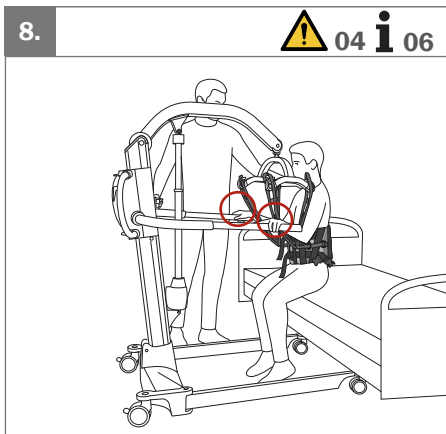
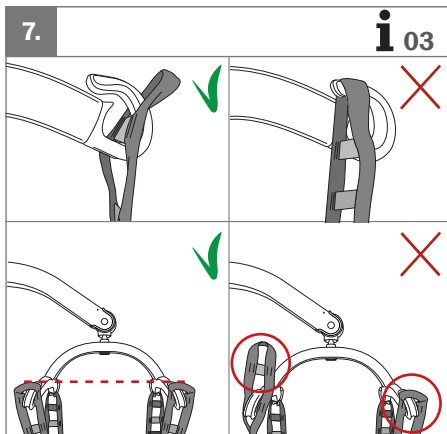
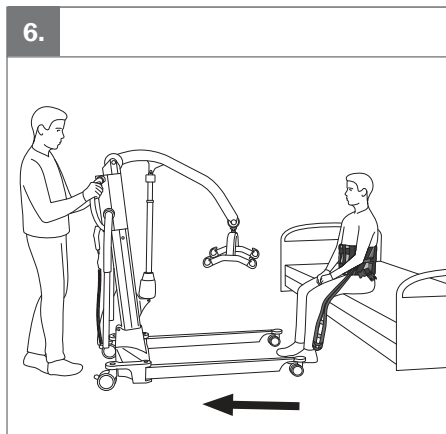
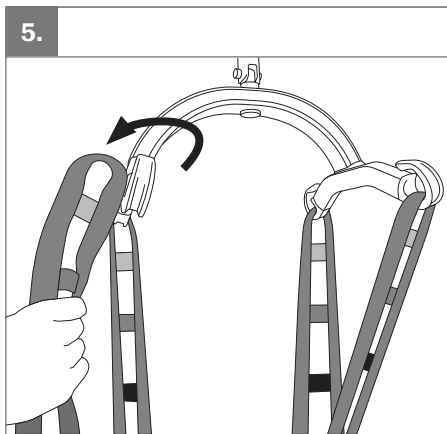
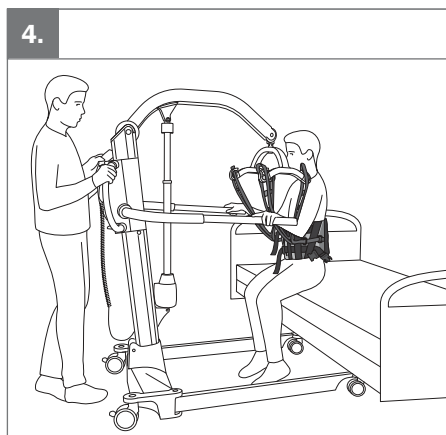
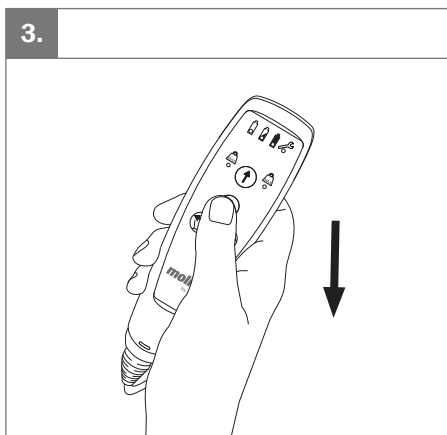
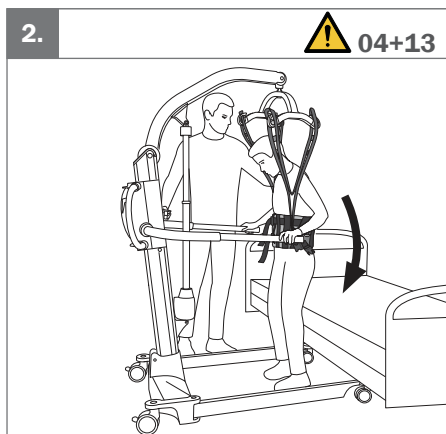
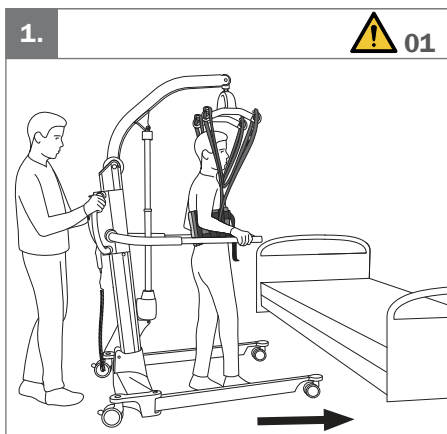
 11

**EN:** Warnings

01. Use the push handles to move the hoist.
02. Avoid deep pile carpets, high thresholds, uneven or sloping surfaces.
03. Make sure that there is a gap between the motor/actuator of the hoist and the bed.
04. The wheels on a mobile hoist must not be locked during a transfer, unless the user is hoisted from the floor.
05. If elevating bed during transfer, make sure motor/actuator is clear of bed.
06. Be careful when transferring, the user may swing and hit surrounding objects.
07. Never hoist the user higher than necessary.
08. Do not push or pull the user, lifting arm or motor/actuator. This may cause stability problems and hoist may tip over.
09. The hoist shall only be used for movement over short distances. It is not a replacement for wheelchair or similar.
10. Be careful not to damage the hoist when it is transported e.g. to a bed or through a doorway.
11. User must be facing the hoist during a transfer.
12. Remove the sling gently to ensure the user does not roll out of the bed.
13. Make sure not to bump user's head on the sling bar.

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**EN:** Information

01. If needed, use leg adjustment for e.g. to get close to a chair.
02. If needed, elevate the head end of bed before doing the transfer.
03. Make sure the straps are fastened symmetrically on left and right side.
04. Perform a safety stop when the straps are stretched tense. Check that all straps are correctly and symmetrically attached.
05. Adjust the arms to appropriate height e.g. chest height.
06. Instruct the user to grab supporting arms.

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**Etac Ltd.**